**Silk'n**<sup>®</sup>  
Beautiful Technology

# Tightra

H2400

EN

USER MANUAL

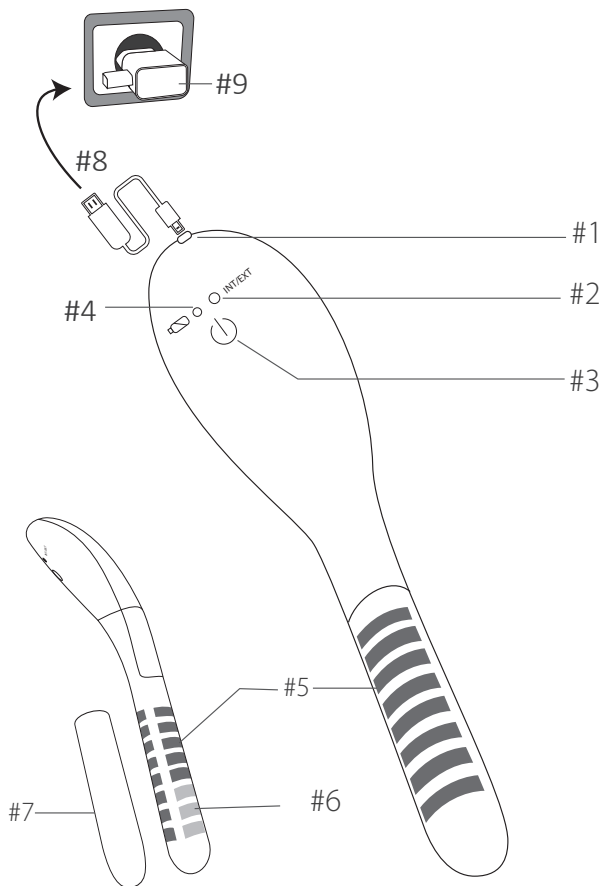
FR

MODE D'EMPLOI

ES

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# Silk'n Tightra

# Table of contents

1. Warnings & contraindications.....	2
2. Signs & symbols .....	8
3. Safety with Silk'n Tightra .....	9
4. Device controls & settings.....	10
5. Get to know your Silk'n Tightra .....	12
6. What to expect with Silk'n Tightra .....	13
7. Possible side effects .....	14
8. Treatment plan.....	16
9. Get started!.....	16
10. Cleaning, maintenance & storage.....	20
11. Troubleshooting .....	20
12. Consumables.....	22
13. Customer service.....	22
14. Warranty .....	22
15. Disposal .....	22
16. Specifications .....	23
17. Labeling .....	24
18. EMC Technical results .....	25
19. Legal notice .....	26

## 1. Warnings & contraindications

---

EN

A statement or event that indicates a possible or impending danger, problem, or other unpleasant situation when using the device.

### **Electricity and safety warnings**

- Keep the device, USB cable and adapter dry at all times.
- Do not use the device while bathing or taking a shower.
- Always ensure that your hands are dry before connecting the device to the electricity supply.
- Never use the adapter if it has a damaged cable or plug, or isn't working properly, or has been submerged in water. If that happens, unplug the adapter immediately before touching it. Failure to unplug the adapter may result in an electric shock!
- Never pull the adapter out of the socket by the cable. Always remove it by taking hold of the adapter.

- ⦿ Do not operate the device in environments where there is a risk of explosion. Keep the device and all its parts away from open flames and hot surfaces.
- ⦿ Keep the adapter accessible at all times, so it can be removed from the electricity supply easily in case of emergency.
- ⦿ Disconnect the device from the electricity supply after charging.
- ⦿ Never pull the adapter out of the socket by the cable. Always remove it by taking hold of the adapter.
- ⦿ Place the adapter cable in such a way that it does not pose a tripping hazard.
- ⦿ Do not use if the parts or accessories are not supplied by the manufacturer as defined in this manual, or if the device is damaged, or does not appear to be working properly. In such cases contact Customer Service.
- ⦿ Ensure there are no unintended rough surfaces, sharp edges or protrusions which may cause harm before each use.

- ⦿ Do not attempt to modify, open or repair the device. You may expose yourself to dangerous electrical components and may cause serious harm to yourself. It will also void the warranty.
- ⦿ The manufacturer accepts no liability for damage or injuries caused by improper or incorrect use.
- ⦿ Children shall not play with the device. Cleaning and user maintenance shall not be made by children.
- ⦿ Children must not play with the packaging. They may injure themselves with the packaging material, or swallow small parts and choke.
- ⦿ If your device reaches its end of life stage, take the device and all its parts to a recycling point designed for electrical devices. Refer to section 15. Disposal.

## **Contraindications**

This device including all its parts must not be used by the following people: children, transgender women, adults with limited physical, sensory or psychological capacities, or those lacking experience or knowledge in how to use the device in a safe way, or

those who do not understand the hazards involved.

### **Do not use if you:**

- ⦿ are pregnant or nursing.
- ⦿ have had any surgical procedure in the vaginal or pelvic area within the last 12 months, or before complete healing.
- ⦿ have an active electrical implant/device in any region of the body, a pacemaker or internal defibrillator.
- ⦿ have implants in the treatment area.
- ⦿ have severe concurrent conditions, such as cardiac disorders, sensory disturbances, epilepsy, uncontrolled hypertension, liver or kidney diseases.
- ⦿ have a history of skin cancer and genital area cancer, or a current condition of any other type of cancer, or pre-malignant moles.
- ⦿ have an impaired immune system due to immunosuppressive diseases such as AIDS and HIV, or use immunosuppressive medications.
- ⦿ have poorly controlled endocrine disorders, such as diabetes, or thyroid dysfunction and hormonal virilization.

- ⦿ have any active condition in the treatment area, such as sores, psoriasis, eczema, rashes, open lacerations, abrasions, or lesions.
- ⦿ have received treatment with light, laser, RF, or other devices in the treated area within 2-3 weeks for non-ablative procedures, and 6-12 weeks for ablative fractional laser resurfacing (according to treatment severity) prior to treatment, except special recommendations.
- ⦿ have used Isotretinoin (Accutane®) within 6 months prior to treatment.
- ⦿ have a sexually transmitted disease.
- ⦿ have a current urinary tract infection, pelvic or pelvic tract infection.
- ⦿ have received radiation therapy or chemotherapy treatments within the past 3 months.
- ⦿ cannot sense heat properly because of an illness, or other health related issue.
- ⦿ are menstruating.
- ⦿ have any other condition which in your physician's opinion would make it unsafe for you to use the device.

## **Consult your physician before use if you:**

- ⊙ have received an abnormal result from a PAP smear or pelvic exam within the last 2 years.
- ⊙ have a history of diseases stimulated by heat, such as recurrent Herpes Simplex, in the treatment area.
- ⊙ have had vaginal surgery, or you have been told you have a vaginal prolapse.
- ⊙ have a history of skin disorders, keloids, abnormal wound healing, as well as very dry and fragile skin.
- ⊙ have a history of bleeding coagulopathies, or use of anticoagulants except for low-dose aspirin.
- ⊙ have an implanted mesh.
- ⊙ have implanted metal birth control devices in the treatment area.
- ⊙ have used medications, herbs, food supplements, and vitamins known to alter healing within the last two weeks.
- ⊙ have a uterine prolapse, cystocele, or rectocele.

- have received drugs from your physician for a skin condition within the past 6 months.

## 2. Signs & symbols

	Important information about using or maintaining the device.
	Products marked with this symbol fulfil the requirements of the EU directives.
IP23	The device is protected against spraying water when tilted up to an angle of 60°.
	Degree of protection against electric shock: type BF applied part.
	Interference may occur in the vicinity of the equipment.
	Do not dispose with household waste! The WEEE directive requires that you dispose of this electrical device (including all accessories and parts) at a designated recycling collection point for electrical devices.
	Follow the operating instructions.
	Lithium battery inside.

-  This device is not suitable for use in the presence of flammable anaesthetic mixture with air or with Oxygen or Nitrous Oxide.

### 3. Safety with Silk'n Tightra

---

With Silk'n safety comes first. Silk'n Tightra has these safety features:

- **Heat stabilizer:** To prevent overheating, each of the 16 **electrodes** (#5, #6) has a built-in heat stabilizer. When a rise in the treated skin temperature is detected, the stabilizer automatically stops emitting heat, until the skin temperature lowers enough for the stabilizer to resume heating.
- **Touch sensor:** The touch sensor ensures that the Volumetric Heating energy will only pass through the **electrodes** (#5, #6) when the **electrodes** (#5, #6) are in good contact with the skin.
- **Movement sensor:** When you use the device in external mode, the device will only emit energy when you move the device across the skin.
- **Auto shutdown:** This safety feature is activated in one of 3 situations:
  - You have completed a 20 minute treatment session. You will feel 1 vibration.
  - The device has not had firm contact with the skin for more than 2 minutes. You will feel 1 vibration.
  - The battery is too low to complete your 20 minute treatment session.

## 4. Device controls & settings

### 4.1 Device modes

Silk'n Tightra has 2 operational modes: internal and external.

- **Internal mode:** Use this mode to tighten the vaginal tissue. The internal mode has 3 energy levels, low, regular and high.
- **External mode:** Use this mode to tighten the external vulva skin tissue. The external mode has 1 energy level.

### 4.2 Device controls

Control	Mode	Action	Explanation
Control button (#3) 	Internal	Long press	Device activated (internal mode, low energy). The <b>treatment mode indicator light</b> (#2) blinks green; 1 vibration
		Short press	Regular energy
		Short press again	High energy
	External	Short press again	1 energy level only
Treatment mode indicator light (#2)  INT/EXT	Internal	Green (slow blink)	Low energy
		Green (fast blink)	Regular energy
		Green (constant)	High energy
	External	White (constant)	1 energy level only
	Both modes	Green and white (alternating blinks)	System error. The device will auto shutdown

Control	Mode	Action	Explanation
Battery indicator light (#4) 	---	Green (blinks)	Device charging
		Green (constant)	Device charged
	Both modes	Orange (blinks)	Low charge in the battery
		Orange (constant)	No charge in the battery

### 4.3 Changing modes and energy levels

- **Internal mode:** This is the default setting when the device is first switched on.
- **External mode:** To change to the external mode before you start a treatment, short press the **control button** (#3) 3 times. You can do this within the first minute that the device is on.
- **Change modes during use:** Once you are using the device, you will need to shutdown the device, restart it, and then select the mode you want.
- **Change energy levels during use:** To change internal mode energy levels during a treatment, short press the **control button** (#3). Each time you short press, you will cycle through the 3 internal modes.

### 4.4 Device charging

Always charge the device before first use. The initial charge should take approximately 6 hours.

- Place the **USB cable** (#8) into the device. Place the other end of the cable into the **USB adapter** (#9) and connect to the electricity supply. The **battery and treatment mode indicator lights** (#4, #2) will go on and off, and then the **battery indicator light** (#4) will blink green.

- Recharge the device fully after each 20 minute treatment, and check the battery level before you start a new treatment.

## 5. Get to know your Silk'n Tightra

---

### 5.1 Package contents

- Silk'n Tightra device
- USB cable & adapter\*
- Vaginal gel
- User manual
- Warranty card
- Beauty case

\* Varies according to the option purchased.

### 5.2 Device parts

The device has a rechargeable battery which is charged using a standard **USB cable** (#8). Refer to the illustration on page (i) at the front of the manual to see the location of each feature.

- USB port (#1)
- Treatment mode indicator light (#2)
- Control button (#3)
- Battery indicator light (#4)
- Electrodes - internal mode (#5)
- Electrodes - external mode (#6)
- Protective cover (#7)
- USB cable (#8)
- USB adapter (#9)

### 5.3 Device description

Silk'n Tightra is a medical cosmetic device that uses Volumetric Heating Technology (VH). Energy flows between the **electrodes** (#5,#6) and works to promote the formation and reconstruction of vaginal collagen and elastin fibers. This results in firming and tightening of pelvic floor organs, and in the reduction of vaginal symptoms.

## 5.4 Intended use

Silk'n Tightra is an electrically operated device, intended for pelvic firming and toning and improved sexual functioning. The device is intended for self-treatment only, and is not suitable for commercial or therapeutic use. Do not use for any purpose other than as described in this manual.

## 5.5 Reading & storing the user manual

Before first use, please read this user manual in full, and keep it for future use. We recommend you refamiliarise yourself with the instructions before each treatment.

## 6. What to expect with Silk'n Tightra

---

During a Silk'n Tightra treatment session it is normal to experience and feel:

- **A sensation of heat:** both during internal and external treatments.
- **Some redness or slight swelling of the skin:** This may occur around the vagina following an external treatment. These reactions should last no more than 24 hours.
- ❏ If you experience these reactions, we recommend that you apply aloe vera gel on the affected skin.
- **A short vibration:** During a treatment, you will feel a vibration every few minutes. This is normal, and indicates that the device is active.

⦿ **Temporary increase in incontinency:** When using the internal mode, some users might experience the need to pass water more frequently. Don't worry, this will go away in a few days. The effects of incontinency will start to reduce, and you will see an improvement as a result of the treatments.

⚠ Stop use immediately if you see full redness of the skin, blistering or burns.

⚠ Stop use immediately if you have any pain, or if the skin or the device becomes too hot.

## 7. Possible side effects

When used according to the instructions, side effects and complications associated with the use of Silk'n Tightra are uncommon. However, every cosmetic procedure, including those designed for home use, involves some degree of risk. Therefore, it is important that you understand and accept the risks and complications that can occur with systems designed for home use.

<p><b>Minor discomfort</b> (Internal/external modes)</p> <p>Some users may feel mild discomfort during use. This is best described as a mild stinging sensation on the skin around the vagina. The stinging sensation may last during or after treatment.</p>	<p><b>Recommendation:</b></p> <p>Use a lower energy level and wait until you have no discomfort before your next treatment.</p>
<p><b>Excessive heat sensation</b> (Internal /external modes)</p> <p>If you have pain, severe discomfort or a very unpleasant feeling, reduce the energy level until you feel OK.</p>	<p><b>Recommendation:</b></p> <p>Use a lower energy level and wait until the effect has cleared before your next treatment.</p>

<p><b>Allergic reaction</b> In rare cases, you may experience an allergic reaction after the treatment.</p>	<p><b>Recommendation:</b> Stop treatment, and consult your physician if the reaction remains for more than 24 hours.</p>
<p><b>Excessive redness and swelling</b> (External mode)</p> <p>In rare cases, the skin around the vagina may become very red and swollen. The redness and swelling should subside within 2 to 5 days.</p>	<p><b>Recommendation:</b> Treat with frequent applications of aloe vera cream. Wait until the effect has cleared before your next treatment. Consult your physician if the redness does not go away within 5 days.</p>
<p><b>Blisters or burns</b> (External mode)</p> <p>In very rare cases, you may experience some blistering or burning of the skin around the vagina.</p>	<p><b>Recommendation:</b> Cool the affected area and apply anti-burn cream. Wait until the effect has cleared before your next treatment and use a lower energy level. Consult your physician if the skin does not return to normal within a week.</p>
<p><b>Spotting</b> (internal mode)</p> <p>In rare cases, you may experience slight spotting immediately after a treatment.</p>	<p><b>Recommendation:</b> Consult your physician if the effect continues for more than 24 hours.</p>

-  Stop use immediately if any of these effects are more severe.
-  See your physician if the effects do not go away within 2 to 3 days.

## 8. Treatment plan

---

- Each internal or external treatment lasts for 20 minutes.
- You may use the device in one or both modes twice a week, for 4-5 weeks.
- After that, reduce the use gradually to once a week, then to every 2 weeks or less frequently, to meet your specific needs.

## 9. Get started!

---

### 9.1 Before treatment

- Make sure the device battery is not low (see section [4.4 Device charging](#)).
- Empty the bladder before treatment.
- Check that the gel expiry date has not passed.
- Remove the **protective cover** (#7) that is over the **electrodes** (#5, #6) before use.
- Before use, clean the **electrodes** (#5, #6) with a vaginal wipe, and dry with a clean paper towel.
- Never use flammable liquids such as alcohol (including perfumes, sanitizers, or other applications containing alcohol), or acetone to clean the skin before use.
- Remove all piercing or metal from the treatment area before use.

## 9.2 Internal mode treatment

1. Apply a small amount of the gel supplied with the device to all the **electrodes** (#5, #6). There is no need to apply gel internally.
  2. Start internal mode treatments using the low energy setting. Long press the **control button** (#3) once. You will feel a vibration, and the device is now ready for use.
- i** During the treatment, you will feel a vibration approximately once a minute.
3. Lie down on your back and make yourself comfortable.
  4. Insert the **electrodes** (#5, #6) into the vagina as far as is comfortable for you. When the device is inside the vagina you need to make sure that the device is supported during treatments:
    - If your legs are splayed out on the bed, close your legs slightly to hold the device in place.
    - If you lie with your legs bent (and feet on the bed), place a cushion under the device to hold the device in place.
    - You might also wish to hold the device, if you find that helps to keep the device in place.
- i** If after 5 minutes you feel comfortable with the heat, move to the regular energy level by short pressing the **control button** (#3). To use the high energy level, short press the **control button** (#3) one more time.
5. Keep the device in place for 20 minutes. We recommend that you lie still and do not move about. However, it's good to rotate the device from time to time so that the energy keeps flowing across the **electrodes** (#5, #6). After 20 minutes the device will auto shutdown.

6. Remove the device, and wipe the external skin around the vagina with a clean dry towel, or wipes specifically for the vaginal area, to remove any gel residue.

**i** In cases of pain or discomfort:

- If you are using the high or regular energy levels, reduce the energy by pressing the **control button** (#3) 1 or 2 times. Wait 5 minutes for the heat to reduce.
- If you are already using the low energy level, remove the device and wait until you feel OK before using the device again.

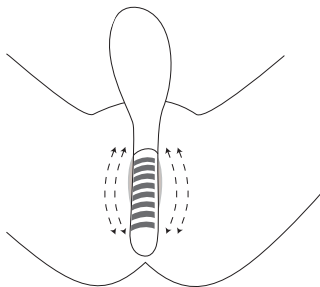
**i** **After your first treatment:** Perform the 20 minute treatment at the highest energy level that is comfortable for you.

### 9.3 External mode treatment

Before use, the skin should be shaven, clean, dry and free of any powders, antiperspirants or deodorants. Also, be sure that the skin is not red or inflamed.

1. Apply a small amount of the gel supplied with the device to the 3 silver-colored **electrodes** (#5, #6). There is no need to apply gel internally.
2. To start external mode treatments, long press the **control button** (#3) and then short press it 3 times. The **treatment mode indicator light** (#2) will be constant white. The device is now ready for use.
3. Lie down and make yourself comfortable.
4. Hold the base of the device and place the 3 silver-colored **electrodes** (#5, #6) on the labia. The triangular shape of the side with the silver-colored **electrodes** (#5, #6) will help you position the device correctly.

5. Keep moving the device slowly in linear or circular motions, covering the outer labia and in between. VH energy will only emit heat, when the **electrodes** (#5, #6) are in good contact with the skin, and you are moving the device.

- i** You can use a mirror to ensure you give enough attention to areas where tightening is most needed.
  - i** During the treatment, you will feel a vibration approximately once a minute.
6. Continue to glide the device across the labia. After 20 minutes the device will auto shutdown.
- i** If you find it difficult to move the device across the skin, you may apply another small amount of gel on the 3 silver-colored **electrodes** (#5, #6).
  - !** If you feel pain or discomfort, remove the device, and wait until you feel OK before using the device again.
7. Remove the device and clean it (see section 10. Cleaning, maintenance & storage).

## 9.4 After treatment

After either internal or external mode treatments, we recommend that you:

- avoid very hot water for 2 days after the treatment.
- contact your physician if there is any indication of infection, excessive swelling, redness, pain, or any other unusual symptom.

## 10. Cleaning, maintenance & storage

---

- Use a vaginal wipe to remove any gel residue off the device including the treatment surface. This prevents any gel from solidifying or damaging the device.
- Then dry the device with a clean paper towel.
- After cleaning, we recommend you store the device in the beauty case. You may wish to put the **protective cover** (#7) over the **electrodes** (#5, #6), but it is not essential.
- ⓘ You may use cleaning fluids with up to 70% alcohol but never use acetone or other abrasive fluids, because they will damage your device.
- ⚠ Never immerse the device or any of its parts in water.

## 11. Troubleshooting

---

Do not attempt to modify, open or repair the device. You may expose yourself to dangerous electrical components and may cause serious harm to yourself. It will also void the warranty.

Problem	Explanation / what to do
My device does not start.	<ul style="list-style-type: none"> <li>• Check that the device is charged. Restart the device.</li> </ul> <p>If the problem persists, contact Customer Service.</p>
I don't feel any heat from the device.	<ul style="list-style-type: none"> <li>• No heat is emitted if the <b>electrodes</b> (#5, #6) are not in good contact with your skin. This is a safety feature. Make sure the device is tightly pressed to the skin before starting treatment.</li> <li>• No heat is emitted if you have not applied gel, or if previously applied gel has dried up. Be sure to apply the gel directly before treatment.</li> </ul>
The green and white lights of the <b>treatment mode indicator light</b> (#2) blink alternately.	<ul style="list-style-type: none"> <li>• There is a system error. Press the <b>control button</b> (#3) to restart the device.</li> </ul> <p>If the problem persists, contact Customer Service.</p>
The device does not charge.	<ul style="list-style-type: none"> <li>• Make sure the adapter is connected to the device and the electricity supply. Unplug the <b>adapter</b> (#9) and then reconnect it again to reset the device.</li> <li>• Check that the <b>USB cable</b> (#8) is not damaged.</li> </ul> <p>If the problem persists, contact Customer Service.</p>
The device has stopped working during a treatment.	<ul style="list-style-type: none"> <li>• Make sure the device is fully charged. The device will shutdown automatically during a treatment if the battery is too low to continue the treatment, or if there is a system error.</li> </ul> <p>If there is a system error, contact Customer Service.</p>
The device has stopped working.	<ul style="list-style-type: none"> <li>• Press the <b>control button</b> (#3) to restart the device.</li> </ul> <p>If the problem persists, contact Customer Service.</p>

## 12. Consumables

---

---

You can purchase more water-based vaginal gel from our website: [www.silkn.com](http://www.silkn.com), or purchase a similar product from other sources.

## 13. Customer service

---

---

See our website for more information: [www.silkn.com](http://www.silkn.com). This manual is also available as a PDF download from: [www.silkn.com/manuals](http://www.silkn.com/manuals). Contact Customer Service to report unexpected operations or events, or any other problem with your device. Our contact

✉ [contact@silkn.com](mailto:contact@silkn.com)    ☎ 1-877-367-4556 (toll-free)

## 14. Warranty

---

---

Please refer to the separate product warranty booklet for complete information.

## 15. Disposal

---

---

If your device reaches its End of Life stage, be sure to follow your country's laws for the disposal of electrical items.

**Packaging:** Dispose of the packaging separated into single type materials. Dispose of cardboard and carton as waste paper and film via the recyclable material collection service.

**Device:** Do not dispose with household waste! The WEEE directive requires that you dispose of this electrical device (including all accessories and parts) at a designated recycling collection point for electrical devices. This ensures that the

device is recycled professionally, and prevents the release of harmful substances\* into the environment. Make sure that the device is in an uncharged state before disposal.

\* Products labelled with Cd = cadmium, Hg = mercury, Pb = lead.

## 16. Specifications

Model	H2400
Technology	Volumetric Heating (VH)
Volumetric Heating (frequency; maximum output power, maximum output current)	1 MHz $\pm$ 30%; 8W max
Operation & safety	Temperature sensor to prevent skin overheating Touch sensor, movement sensor & auto-shutdown for safety
Transport & storage between uses and storage condition	Temperature: - 40 to +70°C Relative humidity: 90%rH max Atmospheric pressure: 500 - 1060hPa
Operating conditions	Temperature: 5 to 40°C Relative humidity: 15 to 90%rH Atmospheric pressure: 700 - 1060hPa
Time from minimum storage temperature between uses to minimum operating temperature with ambient temperature of 20°C	30 minutes
Time from maximum storage temperature between uses to maximum operating temperature with ambient temperature of 20°C	30 minutes
Protection against electric shock	Class 1 medical equipment type BF applied part

Adapter	KSA-10D-05200HU
Adapter input; output	100-240V; 50/60Hz; 0.3A; 5.0V - 2.0A
Service life	5 years

## 17. Labeling

## 18. EMC Technical results

Test	Standard	Class/ Severity Level	Test Result
<b>Emission (IEC 60601-1-2 section 7)</b>			
Conducted emission Freq range 150 kHz – 30 MHz	CISPR 11	Group 1 Class B	N/A
Radiated emission Freq range 30 – 1000 MHz	CISPR 11	Group 1 Class B	Complies
Harmonic current emission test	IEC 61000-3-2	AC mains	N/A
Voltage changes. Voltage fluctuations and Flicker test	IEC 61000-3-3	AC mains	N/A
<b>Immunity (IEC 60601-1-2 section 8)</b>			
Immunity from Electrostatic discharge (ESD)	IEC 61000-4-2	8 kV contact discharges & 15kV air discharges	Complies
Immunity from radiated electromagnetic fields	IEC 61000-4-3	3.0; 10.0 V/m; 80 MHz + 2.7 GHz, 80% AM, 1kHz	Complies
Immunity from Proximity field from wireless communications equipment	IEC 61000-4-3	List of frequencies, from 9 V/m up to 28 V/m, PM (18 Hz or 217 Hz), FM 1 kHz	Complies
Immunity from power frequency magnetic field	IEC 61000-4-8	30 A/m @ 50Hz & 60Hz	Complies

## 19. Legal notice

---

EN

Copyright 2019 © Home Skinovations Ltd. All rights reserved.

Home Skinovations Ltd reserves the right to make changes to its products or specifications to improve performance, reliability, or manufacturability. Information furnished by Home Skinovations Ltd is believed to be accurate and reliable at the time of publication. However, Home Skinovations Ltd assumes no responsibility for its use. No license is granted by its implication or otherwise under any patent or patent rights of Home Skinovations Ltd. No part of this document may be reproduced or transmitted in any form or by any means, electronic or mechanical, for any purpose, without the express written permission of Home Skinovations Ltd. Data is subject to change without notification. Home Skinovations Ltd has patents and pending patent applications, trademarks, copyrights, or other intellectual property rights covering subject matter in this document. The furnishing of this document does not give you any license to these patents, trademarks, copyrights, or other intellectual property rights except as expressly provided in any written agreement from Home Skinovations Ltd. Specifications are subject to change without notice. Silk'n and the Silk'n logo are registered trademarks of Home Skinovations Ltd, Tavor Building, Shaar Yokneam P.O.Box 533, Yokneam 2069206, ISRAEL.

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Tous droits réservés. Copyright 2019 © Home Skinovations Ltd. Tous droits réservés.
Derechos de autor 2019 © Home Skinovations Ltd. Todos los derechos reservados.